



MISSOURI DEPARTMENT OF HEALTH AND SENIOR SERVICES STATE PUBLIC HEALTH LABORATORY

ALCO-SENSOR IV WITH PRINTER MAINTENANCE REPORT

REPORT #7

Complete this report in duplicate at the Send copy to Department of Health and	ime of the regular monthl Senior Services; retain or	y preventative iginal in depar	maintena	nce check, ar	nd wher	never instrument is	repaired
ALCO SENSOR IV SN /// (% TO LOCATION OF INSTRUMENT (STREET AND CITY)	NAME OF AGENCY				1	OF INSPECTION	······································
CHECKLIST: Place a mark in the box by	ร่งก each item if found to be se	tisfactory or if	oporation		TIME	OF INSPECTION	
The state of the s	a de corrected before Usir	ng instrument.		within establish	ned limi	its. (Write in observ	ed values
✓ DIGITAL READOUT (ALL ELEMENT							
PRINTER WORKING PROPERLY	1 (10 0 - 40 0)				<u></u>		
TIME AND DATE DISPLAYING PRO	PERLY						
BREATH ALCOHOL ACCURACY STAN					· · · · · · · · · · · · · · · · · · ·		
SIMULATOR SOLUTION		СОМІ	PRESSED	ETHANOL-G	AS MIX	TURE	
STANDARD SUPPLIER	L	LOT#_ <u>2/</u>	080	_ EXP. DATE	3-	8-23	
SIMULATOR TEMPERATURE (34°C	± 0.2°C) <u>73. 98</u> s	IM. SN _ M	P 35-73	SIM. 1	VIST EX	XP DATE 8-5-	27
less. Check the box corresponding to 0.100% STANDARD - MUST RE 0.080% STANDARD - MUST RE 0.040% STANDARD - MUST RE	AD BETWEEN 0.095% ar AD BETWEEN 0.076% ar	nd 0.105% IN(nd 0.084% IN(CLUSIVE				
TEST 1 ** ,	TEST 2 * , io.	<u>/</u>	TI	EST 3 🖝	.101		
RFI DETECTOR OPERATING							
INDICATE THE NUMBER OF BREATH TO (DO NOT INCLUDE SELF-ADMINISTERE	ESTS IN THE FOLLOWIN D TESTS)	IG RANGES	SINCE TH	E LAST MAIN	TENA	NCE REPORT:	
REFUSALS Ø (004) Ø	(.0509)	(.1014)	1	(.1519)	/	(OVER .19)	6
List any new parts and describe any alteral established limits (use other side if necess.	ary).	was made to	restore the	e instrument to	opera	te satisfactorily an	d within
			w.	···			
INSPECTING OFFICER				Special	aren pos		
SIGNATURE			PAI	VT NAME	25.8	riabilitaria in interpreta di Austria	THE MANY
TYPE II PERMIT NUMBER/EXPIRATION DATE				Mehaz,	<u>/ Ce</u>	Nunel	
200163/5-4-22			1.		- 48	38	
Return completed report to the: Breath aby mail,	Alcohol Program, MO Dep fax, or email.	artment of He	ealth and S				e

Bouchen	AS IV Serial no: 111650 Uersion no: 532B TEST RECORD 00171 Femme Date Time 210L Air Blank: 58/31/21 13:52 .000 Calibration Check: 24 08/31/21 13:52 .101 Subject Name Subject Name Subject I.D. Jest / Operator Name: I.D. Location STY & P.M. St
Bourbin	AS IV Serial no: 111650 Uersion no: 532B TEST RECORD 00172 Temp Date Time 2101 Air Blank: 68/31/21 13:54 .000 Calibration Check: 25 08/31/21 13:54 .101 Subject Name \$lan \(k \) Subject I.D. Querator Name: 1.D. Cen/Un2/ \(20/65 \) Location
Bowler	AS IV Serial no: 111650 Version no: 532B TEST RECORD 90173 Temp Date Time 210L Air Blank: 88/31/21 13:56 .000 Calibration Check: 25 88/31/21 13:56 .101 Subject Name Subject Name Subject Name: I.D. Cardun: 200/63 Location
Enw tom	AS IV Serial no: 111658 Uersion no: 532B TEST RECORD 88174 TEMP Date Time 218L UOID: RFI 12 88/31/21 13:58 Subject Name Subject I.D. PF Operator Name, I.D. Location RSS F fac SI



CERTIFICATE OF ANALYSIS

Certified Alcohol Reference Solution for Simulator

Random Samples of Lot Number 21080 of Alcohol Reference Solution for Simulator were analyzed by gas chromatography on March 10, 2021, using a Perkin Elmer Gas Chromatograph Autosystem XL S/N: 610N9030209, and found to contain 0.1214% (w/vol) ethyl alcohol. The expiration date for this lot number is March 8, 2023 at 11:59 PM.

When used in a calibrated Simulator, operating at 34°C +/- .2°C, this solution will give a breath alcohol analysis instrument reading of 0.100 g/210L +/- 3%.

The alcohol and water used in this solution were free of test interfering substances.

Ted L. Pauley, President GUTH LABORATORIES, INC.

NIST Traceability:

Testing was conducted using Cerilliant Reference Standard lot number FN03052002 whose values are traceable to NIST.

All balances are calibrated annually by an outside agency using NIST traceable weights. Calibration verification is done prior to each use utilizing NIST traceable weights.




STATE OF MISSOURI DEPARTMENT OF HEALTH AND SENIOR SERVICES BREATH ALCOHOL PROGRAM



PERMIT TYPE II

MICHAEL S CENTUNZI

in trereby sufficienced to instruct and superview operators, for and operate the following breath analyzar(a): ALCO-SENSOR:	ain instructors, inspect, calibrate, perform field service and repairs, IV WITH PRINTER
for the determination of the alcoholic content of blood from a 577,020 through 577,041, RSMo and 306,111 through 306.	sample of expired six Permil issued under the provisions of sections 119 RBMs.
DATE	DIRECTOR OF BYATE PUBLIC HEALTH LABORATORY
MANUSER 280163	
MD Min MOTI (N-MA	DIMETERS OF DEPARTMENT OF HEALTH AND REPROPER

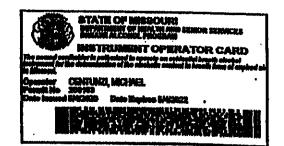

BREATH ALCOHOL TESTING LOG

_	т	 	<u> </u>	 -		 	 	 	,	 	·,-	 	 	 					•
					ū.											8/7/21	C8/07/21	1 2	Date
																STERRIMAN, CHANCE	10 721 Mathis Chanse	ond sort willie.	Subject Name
																LEACH 1240	C.B. Tainsend	Autesmig officer	A mosting offi
																LEACH	Constunct	lesting officer	
				•													,146	result	- - -
																FZRSY	DUI 1st	Comments	